ES - EN

W002W  
Manual de instrucciones  
Instruction Manual  
WE 2019-W002W

# CALENTADOR DE CERA (2 CARTUCHOS) DOUBLE-ROLLER WAX HEATER

## ESPAÑOL

### ADVERTENCIA

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## ENGLISH

### WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• Don't connect the power when there is no wax in the wax machine.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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### WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

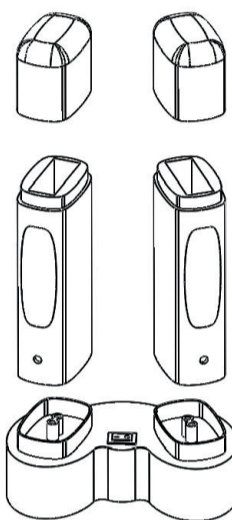
The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

### PACKAGE CONTENTS

• These components are subject to change without prior notice.

• Appearance of product may differ from the image.

No.	PRODUCT	REF.	QTY.
1	APPLICATOR	-	2
2	BASE	-	1

### INSTRUCTIONS OF USE

**1.** Insert the wax cartridge into the melting wax device inner housing.

**2.** Plug the wax heater to the main supply.

**3.** Heat the wax for about 20 or 30 minutes. Then, you can start using the hair removal device.

**4.** Ensure that the skin is absolutely clean before its use.

**5.** Scroll the product following the direction of the hair growth covering an area of 15-18 cm. Before starting to use it, ensure the wax reaches the right temperature. If the wax is too hot, wait for a while before it cools a bit down.

### TECHNICAL SHEET

Weights and sizes	Packing size (cm)	18,5x15,5x24
	Gross weight (kg)	0,72
	Product size (cm)	17x10x25
Electrical characteristics	Rated voltage	110V ~ 230V
	Frecuency	50Hz/60Hz
	Rated power	26 W

**6.** Use a hair removal paper strip to remove the hair. Press it with the palm of your hand on the skin.

**7.** Pull it off quickly in the opposite direction of the hair growth.

**8.** Once you finish, use a hydrating lotion or a moisturizer to massage the skin.

### CAUTIONS

- Turn off the power before cleaning.
- Use a soft cloth, dipped into a soluble wax cleaning product, to remove the residual wax from the device.
- Do not soak it into water.
- Do not use this device on particularly sensitive skin or if the patient has other skin problems.
- Avoid the sun and ultraviolet radiation within 24 hours after the hair removal.
- Use the device under safe conditions and make responsible use of it.
- Do not leave children unattended with this device.
- If the power cord is damaged, buy the component at the manufacturer or at an authorised technical service.
- Once the wax has been heated, please wait for a whole before applying it onto skin.

### MAINTENANCE

• Turn off the power when the machine is not being used.

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© 2018 Weelko