Zoom. Ref 1006

Manual de instrucciones  
Instruction manual  
Mode d'emploi  
Bedienungsanleitung  
Manuale d'istruzioni

Lámpara lupa led  
Magnifying lamp leds  
Lampe loupe à leds  
Led lupenleuchte  
Lampada lente led

---

## Advertencia

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## Warning

- Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components of your unit.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

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## EC-Declaration of compliance

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU.
2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU.

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## Recycling

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

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### Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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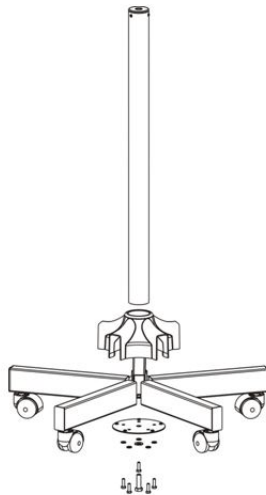
## Instructions of use

1. Assemble the base according to the sketch shown below:
2. Insert the lamp into the base.
3. Connect the power supply.
4. Turn on "POWER" button.
5. Open the plastic cover and align the magnifying lamp with the treatment zone.
6. Adjust the distance between the lamp and the skin.
7. Adjust the brightness to a good level.
8. Once the distance is properly adjusted you can proceed with skin with the skin analyze or other treatments.

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### **Cautions**

- During treatment, if brightness needs to be adjusted, please make sure hands are dry and avoid liquid goes into adjustment button.

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### **Maintenance**

- If there is dust on the lamp glass, clean it with a special soft glass paper and no other material, because it could give damage the glass.

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## TECHNICAL SHEET

Weights and sizes	Packing size (cm)	83,5x69,5x13,5
	Gross weight (kg)	12
	Net weight (kg)	10
Working characteristics	Magnification (diopters)	5 D
	Number of leds (SMT)	20
	Lighting (lux/lumen)	2500 / 420
Electrical characteristics	Rated voltage	220V ~ 240V
	Frequency	50Hz/60Hz
	Rated power	8 W

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