ES Manual de instrucciones
EN Instruction manual
FR Mode d'emploi
DE Bedienungsanleitung
IT Manuale d'istruzioni

WKT001

SPIN

CARRITO CON CAJÓN UV
TROLLEY WITH UV DRAWER
GUÉRIDON AVEC TIROIR UV
Weißer Wagen aus Sperrholz
CARRELLO CON CASSETE UV

USO DEL MANUAL

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USE OF THE MANUAL

It is important that you read and retain this user manual for future reference. It contains important information on its use and maintenance.

You can also ask a professional for guidance to make sure you are using the bed correctly.

WARNING

- Never under any circumstances attempt to open or inspect the internal components of the device. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.
- Never use the device in humid or poorly ventilated environments, such as wet rooms, bathrooms etc. to prevent damage to the electrical components.
- Never use the unit outdoors; exposure to the elements can cause damage.
- If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards or contact the dealer for service.

EC-DECLARATION OF COMPLIANCE

This product meets the following directives:

- Low Voltage Directive: 2014/35/EU
- Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve natural resources and improve standards of environmental protection regarding the treatment of electrical waste.

WELKOME!

Thank you very much for your purchase. Please read carefully the instructions before using the device. We accept no responsibility for damages caused by misuse.

The company reserves the right to amend the details of the devices with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

CONTENT

On arrival of the package, check the presence of the following items:

- Trolley and components

OPERATION

- Connect the power cord into the power supply.
- Place one or two devices on the pull-out anti-collision shelf.
- Store your tools and items into the two standard drawers and use the one on the top to sterilise them.
- The UV drawer contains a mirror to allow 360 degrees of disinfection- You need to keep the tools inside around 15 minutes to a proper sterilisation process.

FIXED AND MOBILE WHEELS

DRAWERS AND ANTI-COLLISION TRAY

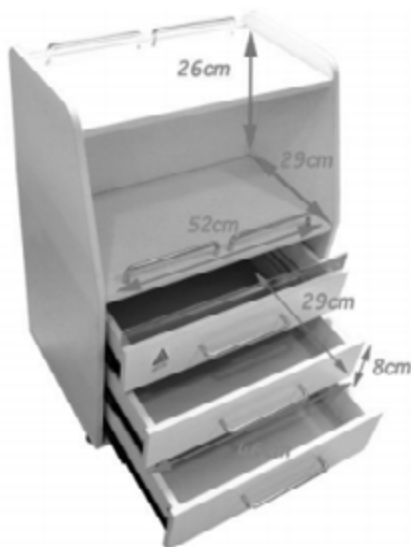
REAR PART

DIMENSIONS

WARNINGS

1. Pay attention when dragging the trolley not to damage the wood.
2. Avoid exposing the product to the sunlight.
3. Please note that the ultraviolet light of the UV drawer automatically stops once you open it to protect the person using it from the UV rays.

MAINTENANCE

1. Use a damp cloth with some drops of soap to clean the surface. Dry it immediately to prevent the wood from getting wet.
2. Use a disinfection product to clean the UV drawer after each use.

CHANGE OF FLUORESCENT

1. Take out the drawer first. To remove it, we must press the tabs that we find in the guide, these are the ones that hold the drawer. There is a tab in each guide, one should be pressed up and the other down. As indicated in the photo.

2. When you take the drawer out, you can see the fluorescent support. You do not have to change the support, you just need to change the lamp.

3. Make half a turn on the fluorescent and it will come out easily.
 4. Place the new lamp with a half turn.
 5. The product is now ready to be used.
- Part ref: 10071 (support and lamp)**

TECHNICAL SPECIFICATIONS

RATED POWER	6 w
VOLTAGE	AC220V
FFREQUENCY	50 Hz
NET/GROSS WEIGHT	26/33 KG
DIMENSIONS (LXWXH)	55 x 40 x 53,5/70
MAX. LOAD	50 kg
MATERIAL	White Plywood

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© 2020 Weelko