Fast +. Ref 1023AB2

Manual de instrucciones
Instruction manual
Mode d'emploi
Bedienungsanleitung

TABURETE
STOOL
TABOURET
HOCKER

¡Bienvenido!

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Welcome!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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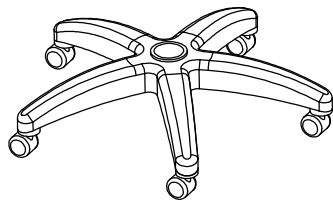
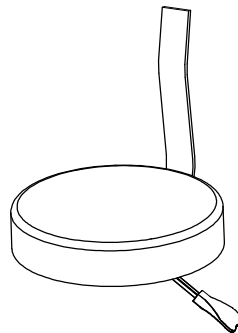
1x

Package contents

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

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
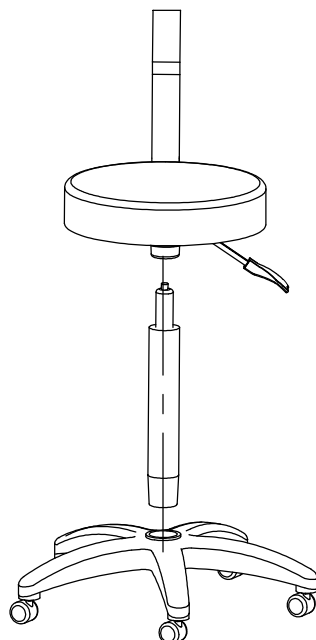
Assembly

1. Insert the piston on the basis of 5 arms.
2. Place the backrest into place and tighten the screw that fixes the position.

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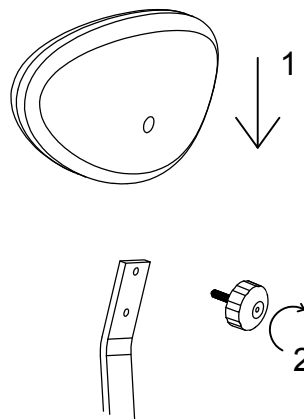Instructions of use

- To raise the stool, pull the lever up, without sitting on the seat.
- To lower the stool, pull the lever up, sitting on the seat.
- To adjust the backrest height, loosen the screw and fix it in one of two positions marked on the plate.

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Caution

The chair has been designed to fulfill its function effectively. It should not be used for any other purpose.

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Maintenance

Avoid exposing the chair to direct sunlight.
Cleaning of upholstery: dye stains should be removed immediately with water.
Cleaning of metal: use a multipurpose cleaner. Do not use bleach or ammonia-containing products.

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