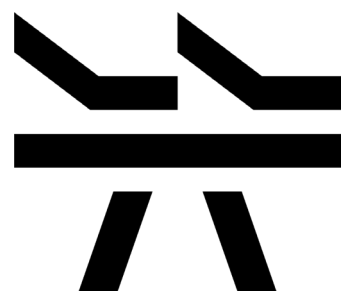
naggura'

Stool with backrest.Ref 1025B

Manual de instrucciones
Instrucion manual
Mode d'emploi
Bedienungsanleitung
Manuale d'istruzioni

TABURETE
STOOL
TABOURET
HOCKER
SGABELLO

¡Bienvenido!

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Welcome!

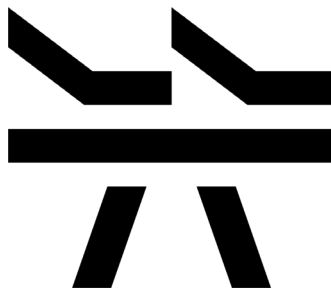
Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.

The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

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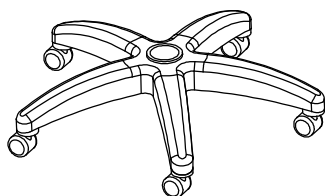
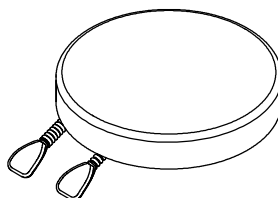
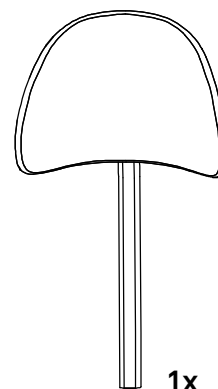
1x

Package contents

- These components are subject to change without prior notice.
- Appearance of product may differ from the image.

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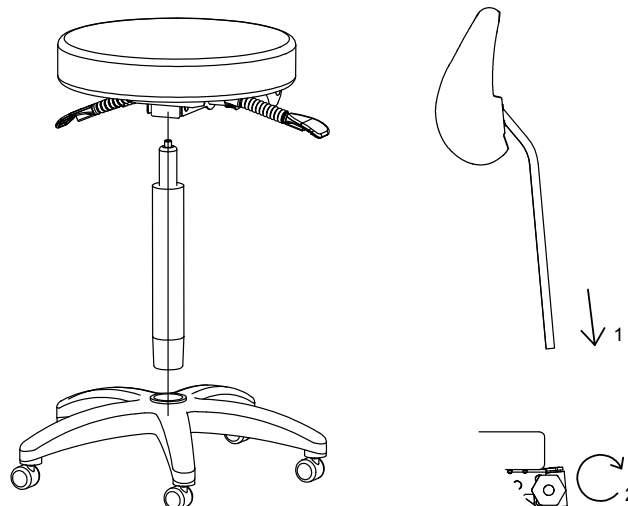
Assembly

1. Insert the piston on the basis of 5 arms.
2. Place the chair on the pump. The piston of the pump must be correctly inserted into the hole of the metal plate under the chair so that it is locked in place.

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Instructions of use

- To raise the stool, pull the lever up, without sitting on the seat.
- To lower the stool, pull the lever up, sitting on the seat.
- To adjust the angle of the seat, pull the lever on the left of the seat up. The seat is free.
- To adjust the backrest angle, pull the lever on the right and back up. The backrest is free.
- To adjust the backrest height, unscrew the screw, adjust the backrest to the desired height and then tighten the screw again.
- To fix the desired position pull the lever down.

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Caution

The chair has been designed to fulfill its function effectively. It should not be used for any other purpose.

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Maintenance

Avoid exposing the chair to direct sunlight.
Cleaning of upholstery: dye stains should be removed immediately with water.
Cleaning of metal: use a multipurpose cleaner. Do not use bleach or ammonia-containing products.

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