ES

WKB004
Manual de instrucciones
WE-2020-WKB004-1

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ADVERTENCIA

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RADIOFREQUENCY + CAVITATION

WARNING

• Never under any circumstances attempt to open or inspect the internal components or accessories of the machine. If an inspection becomes necessary, please contact your supplier or a suitably competent technician.

• Never use the unit in humid or poorly ventilated environments, such as wet rooms, bathrooms, etc. to prevent damage to the electrical components of your unit.

• Never use the unit outdoors; exposure to the elements can cause damage.

• If the power cord was destroyed, please ask a professional to service and replace it to avoid hazards, or contact the dealer for service.

• Do not use on pregnant women, on people with pace-makers (any electrical or rhythmic heart-regulating device) or with skin infections.

EC-DECLARATION OF COMPLIANCE

This product meets the following requirements:

1. Conditions set out in the Low Voltage Directive: 2014/35/EU

2. Conditions set out in the Electromagnetic Compatibility Directive: 2014/30/EU

RECYCLING

 Whenever any of our electrical products or their presentation packaging bear this symbol, it means that these products should not be treated as conventional household waste in Europe. To ensure proper treatment of this waste, dispose of it in accordance with local laws or as required for the disposal of electrical equipment. This initiative will help preserve save natural resources and improve standards of environmental protection in regard to the treatment of electrical waste.

WELCOME!

Thank you very much for purchase. Please read the instructions carefully before operating the instrument. You can also ask a professional for guidance to make sure you are using the instrument correctly. We will accept no responsibility for any accidents that occur due to incorrect operation.



The company reserves the right to amend the details of the instruments with no prior notice. If any mistakes are found in these instructions, please feel free to send us an e-mail informing us so that we can correct them.

PACKAGE CONTENTS

• These components are subject to change without prior notice.

• Appearance of product may differ from the image.

No.	PRODUCT	QTY.
1	CAVITATION HANDLE	1
2	BIG RF HANDLE	1
3	MEDIUM RF HANDLE	1
4	SMALL RF HANDLE	1
5	MAIN UNIT	1
6	POWER CORD	1

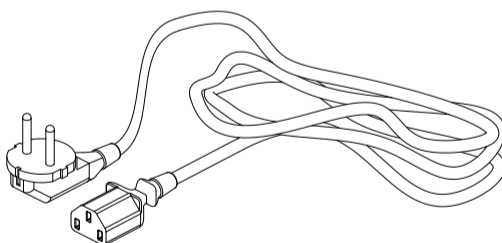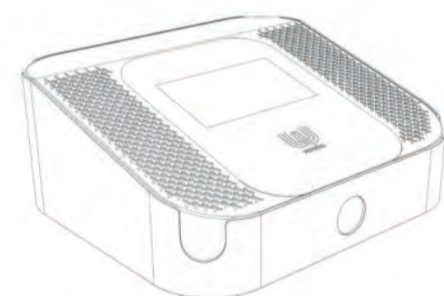
INSTRUCTIONS OF USE

1. Connect the power cord via the rear panel.

2. Connect the accessories and negative plate properly to the rear panel.

3. Turn on the power button on the front panel and click on Weelko logo to select the desired menus.

4. Click on the "face" button if you want to use the medium and small handles for radiofrequency, on the "body" button if you want to use the big handle for radiofrequency and on the "cavitation" button if you want to use the handle for cavitation. The small and medium handles

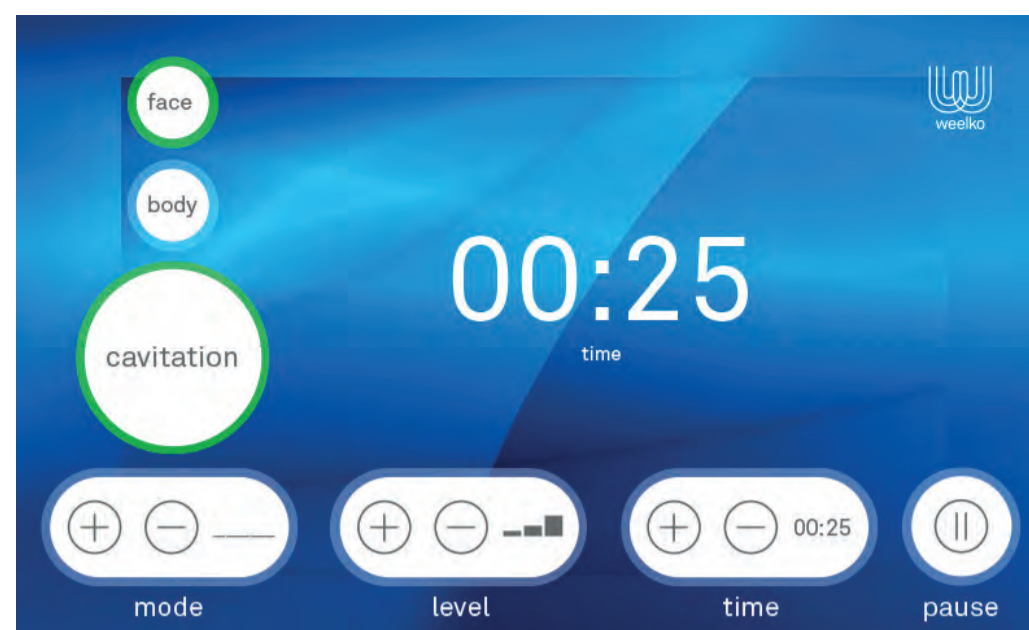use the same connection (4 pins), the big handle uses the other 4-pin connection and the handle for cavitation uses the 2-pin connection.

5. For radiofrequency treatments, use glycerine or specific gel for radiofrequency. For cavitation treatments, transparent conductive gel is recommended, the thickest possible. Apply enough quantity on the patient's skin before putting the electrodes in contact with skin.

6. The "LEVEL" menu allows adjusting 10 intensity levels.

7. The "TIME" menu allows adjusting the treatment time. The default duration is 20 minutes.

8. The "MODE" menu allows choosing between 3 types of wave.

9. Press "PLAY/PAUSE" to begin or to pause the treatment.

MAINTENANCE

• After use, clean the electrodes with a wet cloth

CAUTIONS

• Radiofrequency treatments must not Do not forget to apply gel or glycerin on the area to be treated.

extend over a long period of time. The recommended duration is 10 minutes for facial treatments and 20 minutes for body treatments. In any case, treatments extend beyond 60 minutes.

• Cavitation treatments are recommended on the abdomen, hips, gluteus, thighs and knees. The recommended duration is 20 minutes. In any case, treatments do not extend beyond 40 minutes.

• Apply the electrodes on the customer's skin making constantly circles, without stopping of making movements. Not keep the electrode without moving at the same zone during a long period of time, the customer could be burned.

• During the treatment, check with the patient if the intensity is appropriate.

• The patient and the professional must not wear any metallic objects during the treatment.

• The professional is advised to use insulated gloves to protect hands during the treatment.

ACCESSORIES

No.	PRODUCT
1	CAVITATION HANDLE
2	BIG RF HANDLE
3	MEDIUM RF HANDLE
4	SMALL RF HANDLE
5	MAIN UNIT
6	POWER CORD

TECHNICAL SHEET

Weights and sizes	Packing size (cm)	51x45x22
	Gross weight (kg)	10
	Product size (cm)	34x38x15
Working characteristics	Cavitation frequency	40KHz
	Cavitation power	90W//0-5W/cm2
	Cavitation handle	65cm
	RF frequency	1MHz
	CAVITATION HANDLE	6,8 cm
	BIG RF HANDLE	6 cm
	MEDIUM RF HANDLE	4 cm
SMALL RF HANDLE	2,5 cm	
	Screen	7-inch touch
Electrical characteristics	Rated voltage, Frequency	200-240V, 50/60Hz
	Power	120W

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